


# FIBRICHECK

## YOUR HEART'S HEALTH AT YOUR FINGERTIPS

FibriCheck is a medically certified app capable of measuring your heart rhythm and detecting possible arrhythmias with the use of your smartphone or smartwatch. Your physician prescribed the app to determine the need for further follow-up, possibly related to an upcoming (tele)consult.

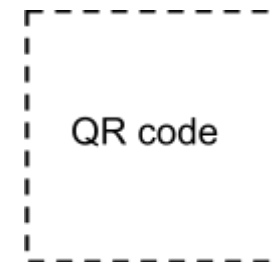

**Invitation code**



### 1 DOWNLOAD FIBRICHECK

Download the FibriCheck app on your smartphone (via the Apple App Store or the Google Play Store).



### 2 CREATE YOUR ACCOUNT

Enter your name, your sex, your birthday and your country. Next, fill in your email address. If you gave your physician your email address, please use that one. After that, choose a password.



### 3 ACTIVATE YOUR INVITATION CODE

Tap 'Start measurement', choose 'I have a QR code' and scan the QR code in the app with your camera or choose 'I have a regular code'. You can find both codes at the top right of this flyer.



### 4 PERFORM MEASUREMENTS

Tap 'Start measurement' again. Place your finger gently on your smartphone camera. After a 60-second measurement your data is sent to your caregiver.



### 5 CHECK YOUR RESULTS

After each measurement you will receive a high level report of your results. All of your data is available for your physician. He or she will contact you in case further follow-up is needed.

**Support needed:** +44 1273 092051 (working hours between 8-10 AM and 1-3 pm) or [support@fbricheck.com](mailto:support@fbricheck.com)

**\*This code has been issued for your use only. You must not share this prescription\code with anyone else.**